


How to use CS550


☞ Switch on the CS550 using the switch in the bag of the screen.



☞ In the APH550 keyboard. Locate the chart control section.

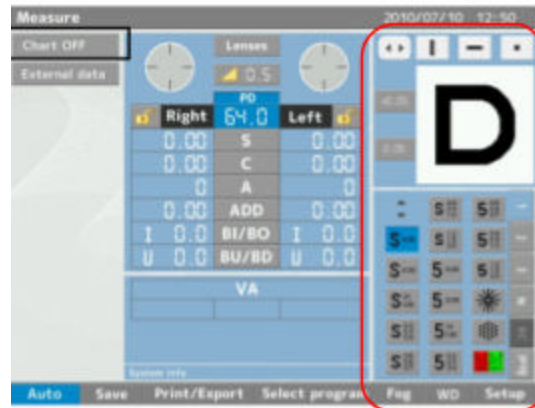

☞ Select a column, a line or a single letter.

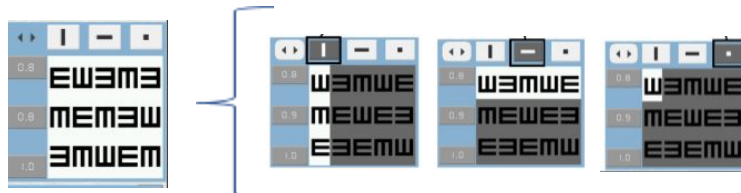

☞ Select a function. Contrast, Red/Green and Reverse background

Change the Contrast



Use of Red/Green



Reverse background




Instructions to use the remote control.


☞ Recharge the remote control. Connect the USB cable to remote control bottom USB port. The other end of the cable connect it to a regular USB charging port.



☞ Remote control buttons navigation

- UP: select this button, can move the key top or left.
- DOWN: select this button, can move the key bottom or right.
- ESC: go to selected features used in the previous step.
- SEL (ENTER): feature of selecting and saving selected feature.

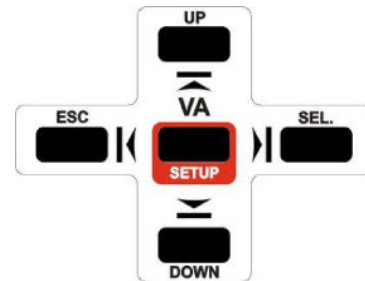


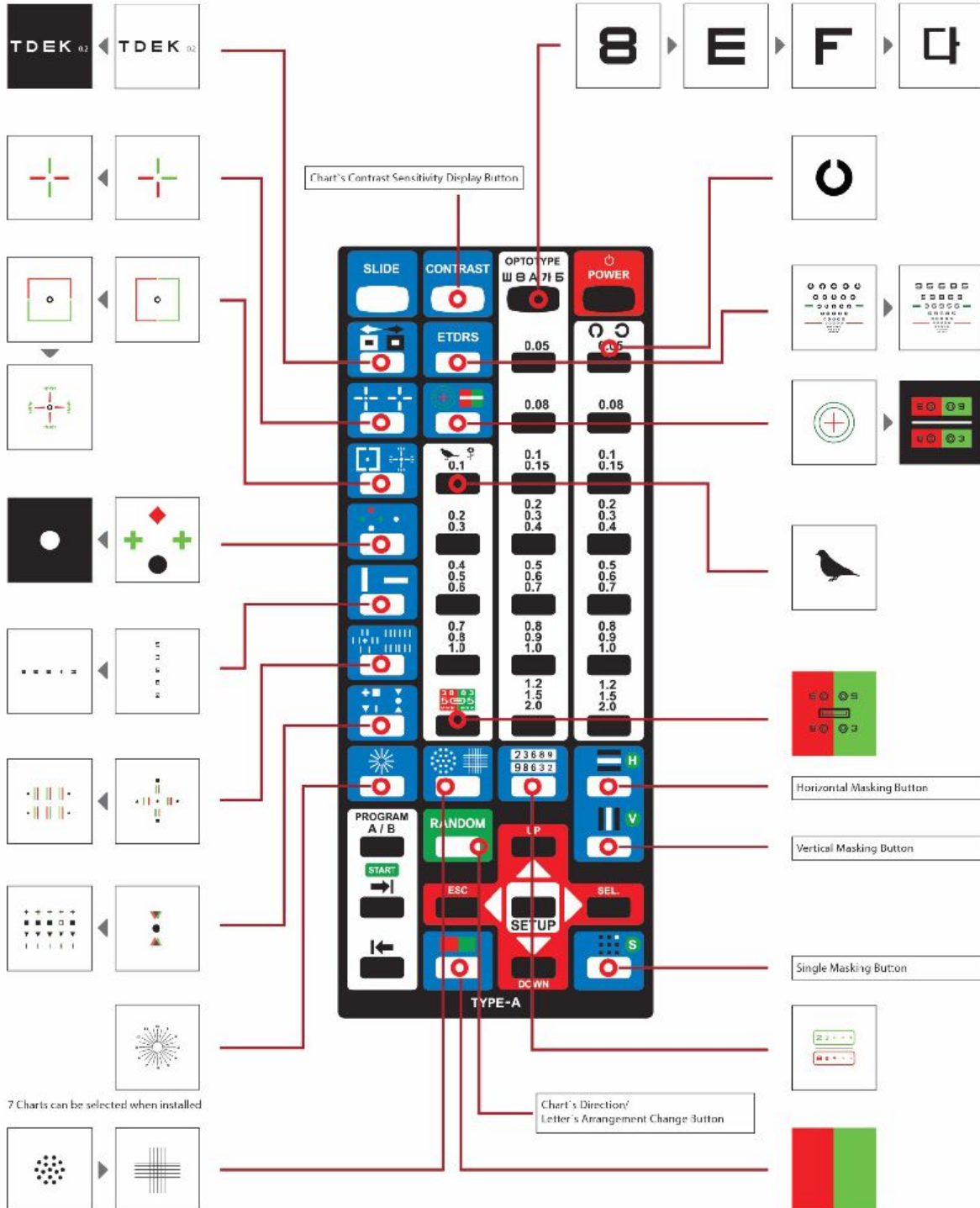

- Select the function after push the slide button




- Eye movement : You can change the movement speed for 3 level

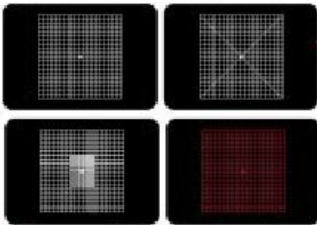



- You can display Amsler grid chart step by step



- You can display Contrast adjustment function for 10 steps

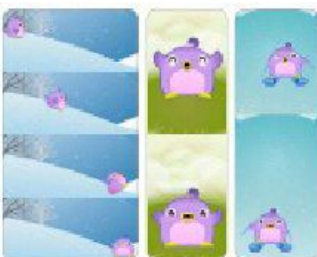

- Animation : When you push the button you can display following image step by step

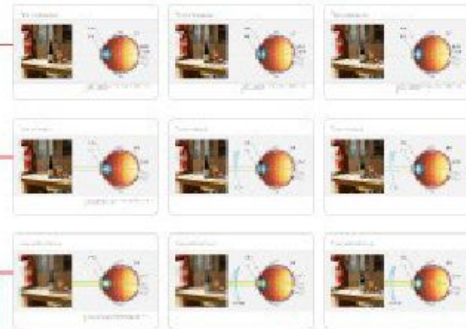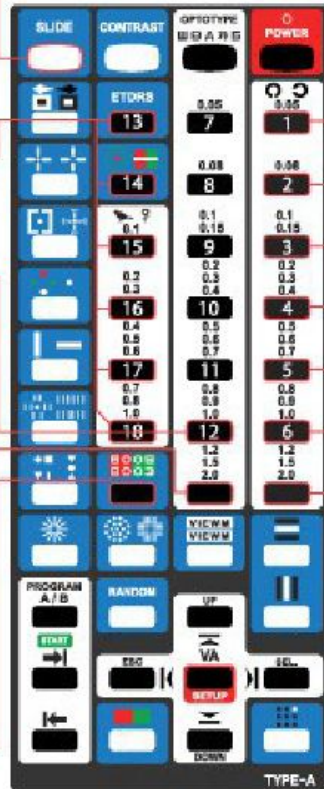

- When you push the slide button chart display a No. 1 key Image. Following Images are
Key 1 : EMMETROPIA
Key 2 : MYOPIA
Key 3 : HYPEROPIA



- Key 4 : ASTIGMATISM
Key 5 : CATARACT - GLAUCOMA - LASIK
Key 6 : Red/Green Stereo chart



- There are 10 kinds of color blindness chart, when you push the button it can be changes step by step