Hand Edge-grinder 32-HANDEDGER

operation instruction



Preface

Dear customers:

Thank you for purchasing 32-HANDEDGER hand edge-grinder. This is your wise choice .In order to help you grasp the correct directions for using the machine and prolong its life of use, you are advised to read this manual carefully.

Part Name:




Specification and power requirement:

Specifications -	Dimensions (L×W×H)	Net Weight
	435 × 315 × 315mm	9kg
	Power Requirement	
	100-120V/60HzAC or 200-240V/50HzAC 120W	


Operation Procedure:



 Remove the water tank case

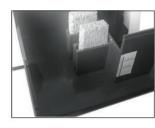

 Add the clean water to the highest level

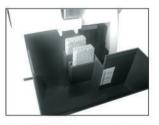

Insert the yellow sponge into water tank



 Close the water tank and cover the water tank case Put down the safe railing. It can be used as soon as turn on the power switch.

Safety Precautions:

- keep the water level of water tank when using .Keep the yellow sponge moist.
 When the electric power can't reach the intensity demand (voltage
- and frequency),don't use the machine.
 It's the attachment of auto lens edger for modifying the lens edge.
- Don't use the machine for any other purposes than hand-grinding the lens edge.